

Setup Guide



Frauenbergstrasse 18 35039 Marburg Germany

Phone: +49(0)6421/1700-0 Email: drg@drg-diagnostics.de

Required Tools:

- 1) 3mm hexagonal wrench
- 2) Disposable working gloves

Procedure:

Disconnect the power cord before removing anything.

Step 1: Facing the back of the machine on the left side, remove the 3 screws, 1, 2, and 3 with the enclosed 3mm hexagonal wrench. (Picture 1)



Step 2: Lift the door up slightly, pushing the side door to the right.



Step 3: Lift the door away from the instrument until you see the hinge below





Step 4: Carefully, remove all six (6) screws with the 3mm hexagonal wrench.

Step 5: Remove the T shaped Metal piece.



Step 5: Slide the interior mechanism forward towards the front of the instrument.






Step 6: Align the T shaped metal back to the plate. Place the 6 screws back into place.



Step 7: Align and prepare to pleace the door panel back on. Align the hinges into the slots and slide the door back into place to the left.



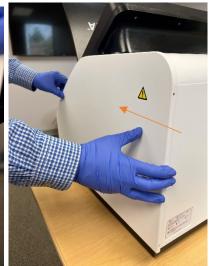

Step 7: Make sure the door is properly aligned on the front seam of the instrument.



Step 8: Align and place the 3 screws back on using the 3mm hexagonal wrench.



Step 9: Make the back plate is aligned and screwed tightly.



Document Revision History		
Version #	Date	Description
1	May 17, 2022	Creation of Document